Ciambella Dolphin
Rubber Ring Cushion

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GOMMA, ACCIAIO, PLASTICA.

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RUBBER, STEEL, PLASTIC.

INDICATION FOR USE:

FOR USE EXCLUSIVELY WHEN PRESCRIBED BY THE
 DOCTOR FOR THE FOLLOWING CASE: FOLLOWING
 HEMORRHOID OPERATIONS PROLONGED BED
 CONFINEMENT OR WHEELCHAIR.

Instructions for use:

Before us, consult with a specialist doctor.
*Before use, verify to make sure the device is in good
 condition, free from cracks, creases and holes.*
*After accurate closure of the inflation valve, clean the
 outer surface of the device using warm water and
 mild soap.*
Inflate moderately.
*Over-inflation can cause breakage and / or
 irreversible damage.*
*To inflate you can use the appropriate pump not
 included in the package, but supplied on request.*
During use a cotton cover is recommended.

Maintenance and disposal:

*Following use, clean accurately, and store the
 product at room temperature, deflated, without
 creases, protected from dust, light and away from
 heat sources.*

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For cleaning do not use abrasive cleaners or detergents (eg. : solvents, acids, alcohol, etc.) that could damage the device.

Carefully wipe the entire surface, in particular the inflation valve.

In case of necessary disposal, follow current legislation.

Cautions and contraindications:

Do not use in direct contact with the skin and / or surfaces injured or compromised.

Do not bend either during or after use.

Avoid contact with sharp objects and / or stinging or, in general, with sharp, angular.

Avoid contact with fire and / or burning objects.

Do not fill with liquids or other substances.

Children under the age of 12, invalids or disabled can not use the device alone, and must be assisted / controlled by other people.

If the device is damaged or compromised, replace it.

Do not use the product differently than as described above.

Technical data

Available diameters: cm 35, 40, 43 and 45.

CE certified according to Directive 93/42 EEC Medical Devices